

EXERCISE MODES

The Companion™ features a number of additional therapy modes to vary and adapt the exercises to the exact needs and progression of the patient.

PATIENT SET UP MODE

One click and the unit will position itself at a non-pain level of 20° flexion (can be adjusted in settings).

EXTENSION/FLEXION MODE

Extension/flexion mode: simple back-and-forth movement over the entire programmed range of motion.

WARM UP MODE

Initiate the session at 70% of ROM and increase by 5% until reaching the full ROM set up.

SPRING MODE

Repeat 3 times the last 10° of extension and flexion (can be adjusted in settings).

EXTREMITY FOCUS MODE

The speed will be minimal during the first 10° and the last 10° of the movement. The speed will be faster during the rest of the range of motion (can be adjusted in settings).

EVALUATION MODE

Three levels of active resistance exercise to test and evaluate patient progress within the rehab process. (The patient performs 5 or 10 repeated exercises during which he is asked to resist against the device whilst going into flexion.)

DYNAMIC STRETCH MODE

Mode used with patients having difficulties reaching full extension. The device will gradually extend the knee with pauses in between each stretching.

TECHNICAL DATA

- Touch screen operated under Android O.S.
- 2 applications: patient version / professional version (optional)
- Multi-lingual software for full patient compliance & understanding
- Bluetooth 4.0 for secure and consistent communication signal
- Wifi connection for remote access
- Possibility to lock the adjustment of the parameters
- Function Start/Stop/Inversion
- 2 year warranty

ORDERING INFORMATION

Kinetic Spectra Companion™	
Deluxe hygienic pads	4621K0600
Comfort supports	4621K0604
Upgrade kit with footplate	4670027282
Upgrade kit without footplate	4670026979
Professional tablet + app	4670027290

Upgrade kit available for Kinetic Spectra™ (knee) and Kinetic Centura™ (shoulder) devices



Professional tablet & app



Kinetic Spectra Companion™



kinetec®

Kinetec SAS
Zone Industrielle de Tournes
Rue Maurice Périn
F-08090 Tournes
France
Tel: +33 324 29 85 05
Fax: +33 324 33 51 05
contact@kinetec.fr
www.kinetec.fr

Kinetic Medical Products UK Ltd
Unit E, Manaway Industrial Estate
13 Holder Road, Aldershot
Hampshire GU12 4RH
United Kingdom
Tel: +44 (0)1252 931980
Fax: +44 (0)1252 329415
enquiries@kinetecuk.com
www.kineticinternational.com

keep aktive™



kinetec®



KOMPANION

Therapy : monitoring : evaluation
THE LATEST IN CPM TECHNOLOGY

KEY INFORMATION


FEATURES & BENEFITS —

The Kinetec Kompanion® technology has been designed to meet the latest healthcare professionals requirements and guide (or accompany) their patients through a unique yet efficient CPM rehabilitation process.

- **DOCTORS**
 - Monitor patient therapy and evolution daily.
 - Real time monitoring on multi patients with the Kinetec Kompanion™ 10.1" professional tablet.
 - Video conference consultation with the patient (video/voice/chat).
 - Data confidentiality.
- **SERVICE PROVIDERS**
 - Create protocols reflecting the doctor prescription.
 - Program the device for the whole therapy time.
 - Set up and education time significantly reduced thanks to the intuitive & easy to use interface.
 - Monitor patient therapy and evolution daily.
 - Real time monitoring on multi patients with the Kinetec Kompanion™ 10.1" professional tablet.
- **PATIENTS**
 - Intuitive dashboard to adjust basic parameters.
 - Functional exercise modes to maximize the therapy efficiency.
 - Multilingual interface for maximum compliance
 - Motivating and guiding pop-up messages during sessions.
 - Daily objectives & achievements on explicit graphics.
 - Educational videos & exercises to improve compliance & outcomes.

EXPLORE KOMPANION —

THE DASHBOARD

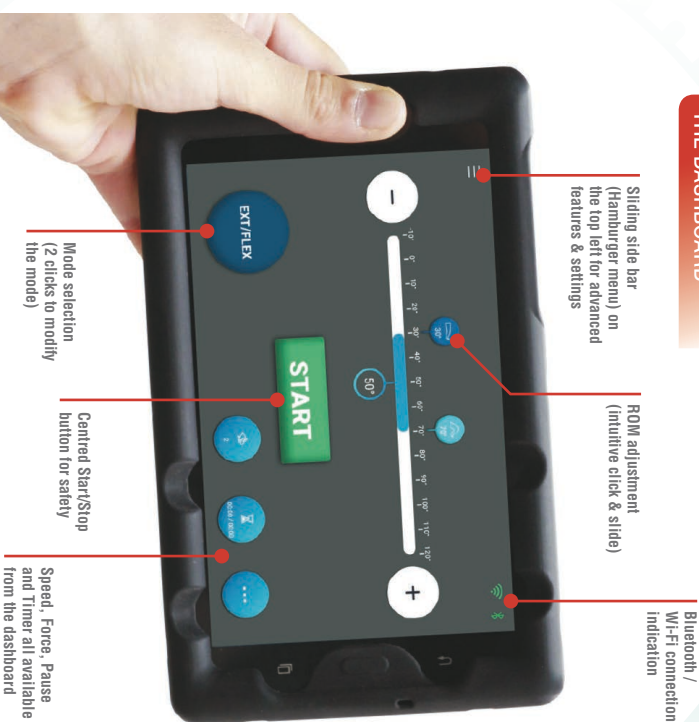

INTUITIVE & EASY TO USE DASHBOARD

- Intuitive tablet for ease of use
- 3 clicks away from most functions
- Multi-lingual app for maximum patient compliance

PATIENT EVALUATION

- Full patient report access
- Monitor & follow up patient progress & compliance with therapy
- Accessible remotely via Kinetec Kompanion™ professional tablet

PROTOCOL CUSTOMIZATION

- Smart functions easily accessible
- Physiological movements replicating therapist manipulations
- Active resistance exercise through 3 levels of evaluations

PATIENT MONITORING

- Monitor an entire patient database
- Live video consultation by the doctor or the rental company
- Program & forward therapy protocols remotely

